


immunocard STAT![®]


HpSA[®]

TEST PROCEDURE


How to perform the test

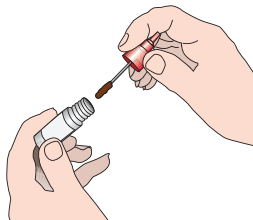

1. Using the applicator stick of the diluent vial, transfer a small portion (5-6 mm diameter) of stool specimen into the sample port. Vortex 15 seconds.

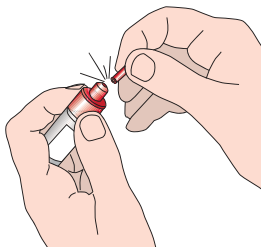


2. Hold the vial vertically and gently tap on benchtop. Break the tip from vial using absorbent paper.

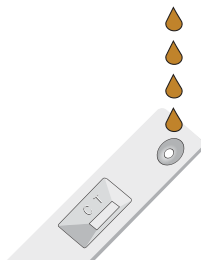


3. Dispense 4 drops into the sample port of the device. After 5 minutes, read result within 1 minute.


Interpretation of Results

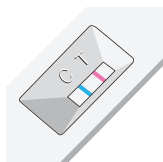

POSITIVE

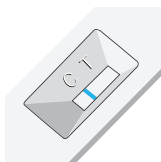

NEGATIVE

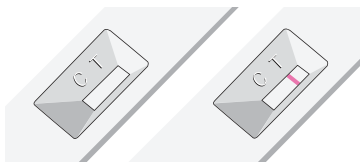

INVALID



This chart does not contain complete instructions for use.
For further information, please refer to the package insert.

USA/CORPORATE OFFICE

3471 River Hills Drive
Cincinnati, Ohio 45244, USA
Telephone: 513-271-3700

EMEA, MERIDIAN BIOSCIENCE EUROPE

Phone: +39 0331433636
E-mail: MBE-TechService@meridianbioscience.eu

Orders/Customer Service: 1-800-543-1980
Technical Support: 1-800-343-3858
Information Fax: 513-272-5432
Ordering Fax: 513-271-0124
meridianbioscience.com

meridian BIOSCIENCE[®]